

iCup[®] 2 Multi-Drug Test Cup




Key Feature:

- **“No Step Operation With Split Specimen”** Once the specimen is added to the cup, screwing on the cap initiates the test. No keys or activation steps are required.

Instant Technologies, Inc.



an  inverness medical company

iCup[®] 2

Multi-Drug Test Cup

Drug Testing Made Simple



Features:

- Available in 5 - 9 Drug Combinations
- Screw Top Lid Eliminates Leaking
- No Keys or Activation Steps
- Split Specimen Sampling
- All Testing Components Included in Kit
- In built Adulterant test, including Creatinine


Test Kit Contents:

- 25 Test cups
- 25 Caps
- 27 Security seals
- 1 Package insert

Test Components:

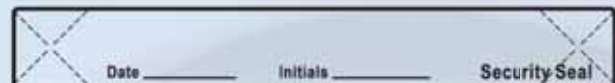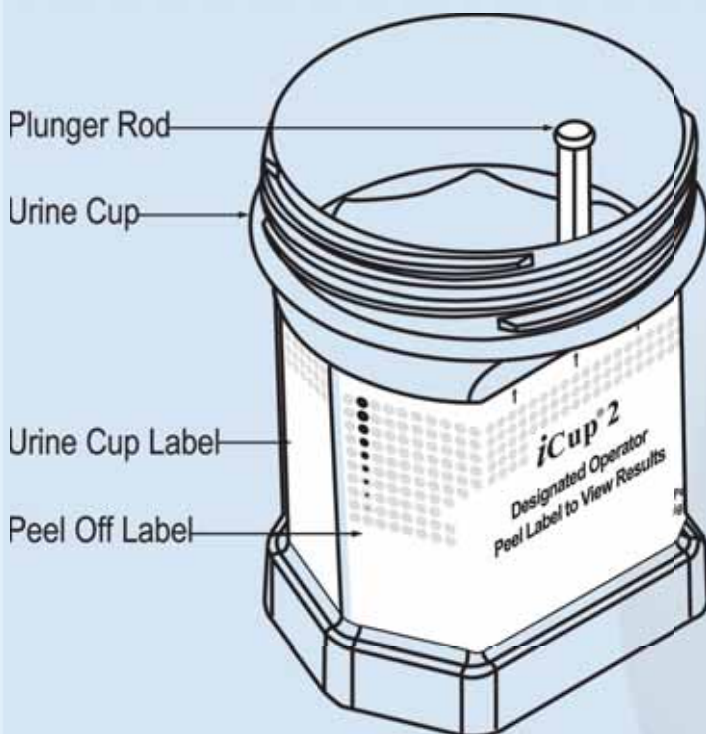

Security Seal

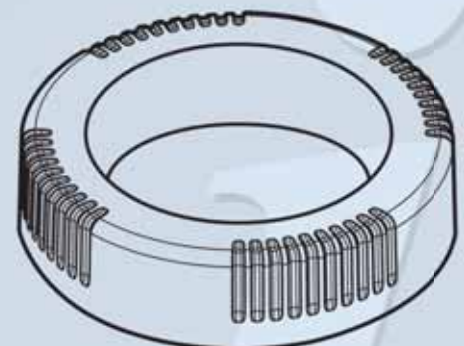

Cap (Included in the foil pouch)

iCup[®] 2 Drug Screen

Drug Testing Made Simple.

Instant Technologies Drugs of Abuse tests are lateral flow chromatographic immunoassays for the qualitative detection of multiple drugs and drug metabolites in urine at the following cut-off concentrations:





Drug Name:

Abbreviation:

Cutoff Levels:

Amphetamine.....	AMP.....	300ng/mL
Methamphetamine.....	MET.....	300ng/mL
Opiates/Morphine.....	MOP.....	300ng/mL
Cannabis.....	THC.....	50ng/mL
Benzodiazepines.....	BZO.....	200ng/mL
Cocaine.....	COC.....	300ng/mL


iCup[®] 2 Drug Screen Procedure

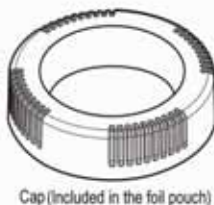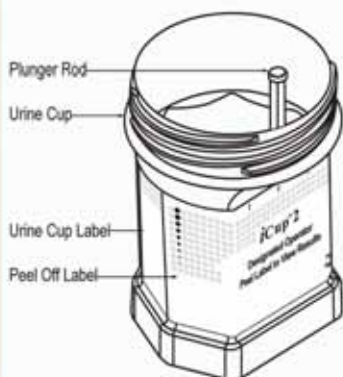


1 Donor provides specimen (Cap is held by operator).



2 The Cap should be screwed on gently but firmly, taking care to properly align the Cap with the threading on the cup while completely depressing the Plunger Rod.

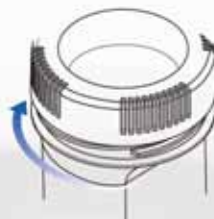


3 Temperature is read between 2-4 minutes to verify range between 32-37°C.

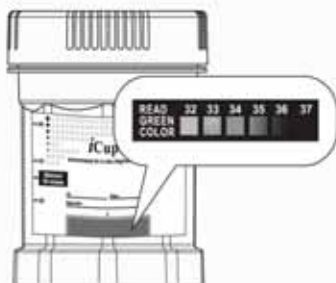


4 Check cap for tight seal. Donor dates and initials the security seal and places security seal over cap as shown.



5 Remove the Peel Off Label covering the test results.

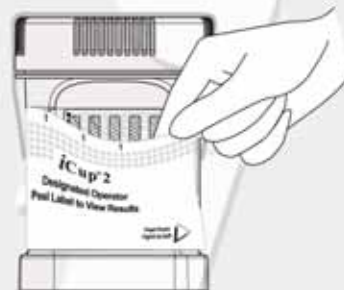

6 Read results.

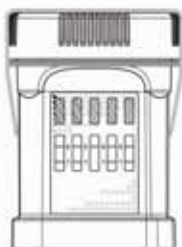

Read the drug strip results at 5 minutes. Test results stable up to 60 minutes.

Negative
 Preliminary Positive
 Invalid

= 5 minutes
= 60 minutes

7 If preliminary positive results are observed, send the cup to the laboratory for confirmation.



Inc.

Product Descriptions & Fact Sheet

DOA Tests: General

<u>Item</u>	<u>Feature</u>	<u>Benefit</u>
Specimen Type	Urine	<ul style="list-style-type: none"> • Simple testing procedure. No special reagents or equipment required.
Time to Results	Results in 5 minutes.	<ul style="list-style-type: none"> • Quick test results. • 60 minute result stability.
Results	<p>POSITIVE: No line appears in the Test region (T) for a specific drug tested. One colored line appears in the control region (C).</p> <p>NEGATIVE: The appearance of a colored line in C region and a colored line in the T region for a specific drug indicate a negative test result.</p>	<ul style="list-style-type: none"> • Lines are easy to read, Excellent line color intensity.
Procedural Control	Built-in procedural control “C” line.	<ul style="list-style-type: none"> • Confirms correct procedural technique.
Accuracy	Accurate results when compared to other commercially available drug test devices.	<ul style="list-style-type: none"> • High correlation to GC/MS.
Test Storage	24 months at room temperature.	<ul style="list-style-type: none"> • Refrigeration of the test not required, valuable refrigeration space remains available for other items.
Tamper Evident	<ul style="list-style-type: none"> • Temperature indicators. • Security seals & labels. 	<ul style="list-style-type: none"> • Reduced tampering.

Key Feature:

- **“In built adulterant test”** – Tests for Creatinine, pH and Oxidants to ensure specimen is a valid urine sample for analysis.
- **“No Step Operation With Split Specimen”** – Once the specimen is added to the cup, screwing on the cap initiates the test. No keys or activation steps are required.

TM

Inc.