


Passive Testing



It is recommended the subject **not consume food, drink or smoke 15 minutes prior to taking a test.**




Press the Test key “**⏻**” to turn the HH3 Alcohol Tester on. Ensure the test type is set to Passive Testing (no mouthpiece). A “**Start Passive Test**” message will display.

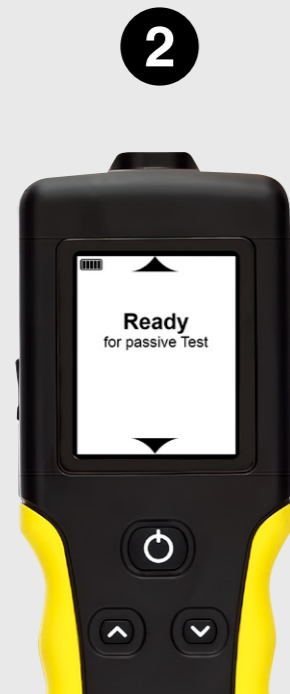

Press the Test key – the screen will display “**Please Wait**” then “**Ready for passive Test**”.

Hold the instrument 50mm in front of the subject’s mouth so that the subject is breathing or talking directly into the device’s inlet.

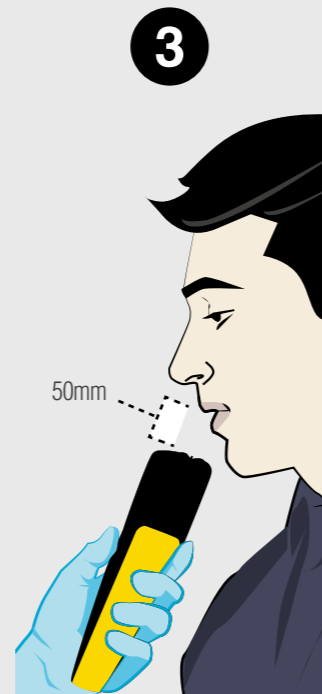

Instruct the subject to count from 20 onwards until you tell them to stop. Whilst they are counting, press the Test key. You will hear 3 clicks.

Tell the subject to stop counting after the 3rd click. The device will analyse, emit a long beep, and then display a text based result.



A screen with text “**Pass No Alcohol**” indicates that **alcohol is not present.**

A screen with text “**Warning Alcohol Present**” indicates that **alcohol is present.**

In the latter instance, change the test type to mouthpiece testing in order to gauge the subject’s BAC level.


Active Testing



It is recommended the subject **not consume food, drink or smoke 15 minutes prior to taking a test.**



Press the Test key “**⏻**” to turn the HH3 Alcohol Tester on. Ensure the test type is set to Active Testing (with mouthpiece). A “**Start Mouthpiece Test**” message will display.

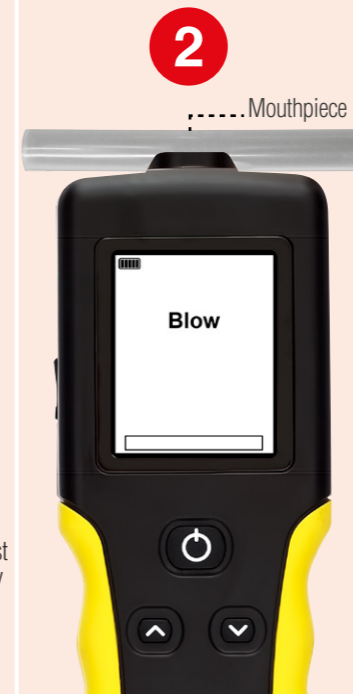

Attach an *Alcolizer approved *mouthpiece. Press the Test key – the screen will display “**Please Wait**” then “**Blow**”.

Hold the breathalyser in front of the subject’s mouth so that the subject can blow into the mouthpiece.



Instruct the subject to blow gently until you tell them to stop. You will hear a sequence of beeps, a click then a long beep to indicate the test is complete.

Tell the subject to stop blowing.

The device will analyse and then display a colour coded numerical BAC result.



A screen with the numerical BAC of “**.000 g/100ml**” indicates that **alcohol is not present.**

A screen with the numerical BAC value of more than “**.000**” indicates that **alcohol is present.** Follow the relevant instructions as designated by local municipality and/or employment terms.