


## Passive Testing

**1**




It is recommended the subject not **consume food, drink or smoke 15 minutes prior to taking a test.**

Press the Test key “**⏻**” to turn the LE5 Alcohol Tester on.

Ensure the test type is set to “**Passive Std**” (no mouthpiece).

A “**press passive test**” message will display.


**2**



Press the Test key – the screen will display “**Please Wait**” then “**Ready for passive Test**”.

Hold the instrument 50mm - 100mm in front of the subject’s mouth so that the subject is breathing or talking directly into the device’s inlet.


**3**

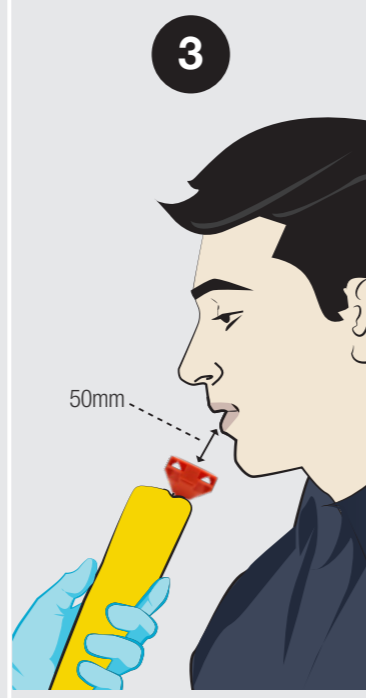

If using pressure activated passive, instruct the donor to blow into the device until you tell them to stop. The device will take a sample automatically. If using button activated passive, instruct the subject to count from 20 onwards in a one second rhythm until you tell them to stop. While they are counting, press the Test key.

You will hear 3 clicks. Tell the subject to stop blowing or counting after the 3rd click.

**4**

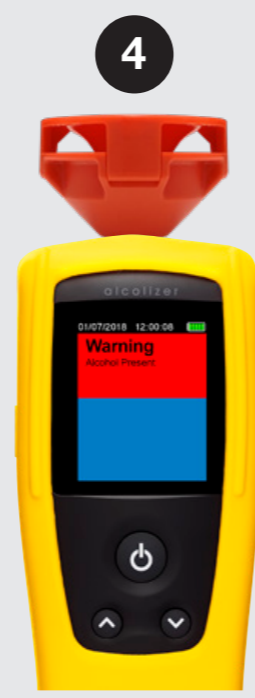

The device will analyse, emit a long beep, and then display a text based result.


**A green screen** with “**Pass No Alcohol**” text indicates **alcohol is not detected.**

**A red screen** with “**Warning Alcohol Detected**” text indicates **alcohol is detected.**

In the latter case, the device automatically switches to a mouthpiece “**Standard**” test displaying “**press for active test**”.

## Mouthpiece Testing

**1**




The subject is **not to consume food, drink or smoke 15 minutes prior to taking a test.**

Press the Test key “**⏻**” to turn the LE5 Alcohol Tester on. Ensure the test type is set to “**Standard**” (with mouthpiece).

A “**press to test**” message will display.

**2**




Attach an \*Alcolizer approved mouthpiece.

Press the Test key – the screen will display “**Please Wait**” then “**Blow**”.

Hold the breathalyser in front of the subject’s mouth so that the subject can blow into the mouthpiece.

**3**

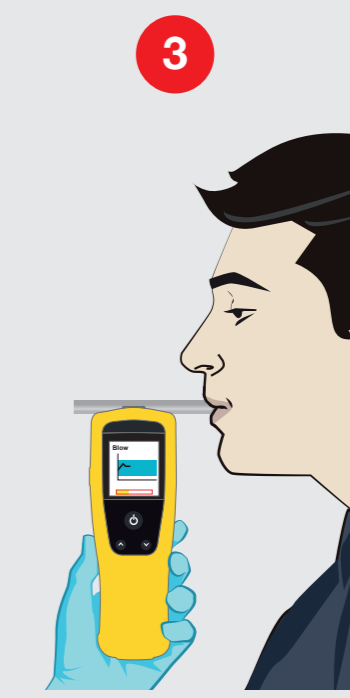


Instruct the subject to take a deep breath and blow gently into the mouthpiece until you tell them to stop.

You will hear a sequence of beeps, a click then a long beep to indicate the test is complete.

Tell the subject to stop blowing.

The device will analyse and then display a colour coded numerical BrAC result.

**4**



**A green screen** with “**0.000g/210L**” BrAC indicates that **alcohol is not detected.**

**A yellow screen** with numerical BrAC indicates that **alcohol is present but under the pre-configured threshold BrAC level.**

**A red screen** with numerical BrAC indicates that **alcohol is detected and is over locally accepted BrAC levels.**