

Securetec DrugWipe Quick Reference Guide



Prior to testing, ask your donor:

Have you had anything to eat, drink, smoke or chew in the last 10 minutes?

Yes – Sit donor aside for 10 minutes and then proceed

No – Proceed

Have you any over-the-counter or prescription medication to declare?

Yes – Record medication and then proceed

No – Proceed

Conducting the test


- 1 Review the foil packaging, taking note of storage conditions, expiration date and batch number. If there is no damage, open it and pull out the DrugWipe. Please remember to retain the packaging until after the test is completed



- 4 While not touching the sampling pads, remove the blue sample collector from the white detection element




- 2 Prior to testing, ensure 3 weak blue lines are visible

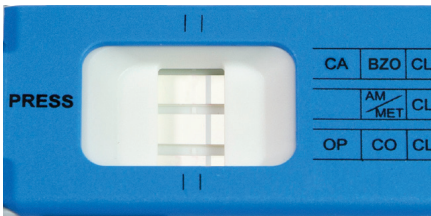

- 5 Advise your donor to move their tongue around the inside of their mouth and cheeks 3 times



- 3 Slide the protective cover in the direction of the arrow until the word 'PRESS' is revealed

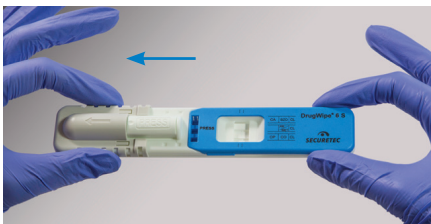

- 6 Check the 3 sample pads on the underside of the blue sample collector are pink



- 7 Hand the blue sample collector to your donor and instruct that they run **ONE GENTLE WIPE** on the 3 sample pads down the centre middle to the tip of their tongue



- 8 Collect the blue sample collector from your donor and ensure all 3 sample pads have changed from pink to yellow



- 9 Reattach the blue sample collector to the white detection element with an audible double-click. Apply a gentle pressure by pinching the device on the blue 'PRESS' mark to ensure everything is fitted correctly



- 10 Bring the DrugWipe into a vertical position with the blue sample collector facing upwards – "Blue to the sky". Crack the buffer fluid ampoule by pressing firmly on the white 'PRESS' mark **ONCE**



- 11 Remove your thumb from the white 'PRESS' mark and keep the device in the vertical position to settle for 10 seconds



- 12 Slide the protective cover back into place

- 13 Place the DrugWipe on a flat surface and allow 8 minutes for the test to complete undisturbed

Interpreting the Results

Negative results = no drugs detected

A negative result indicates there have been no drugs detected. Only all 3 control lines will appear



Non-negative results = drugs detected

A non-negative result indicates there have been drugs detected. All 3 control lines and 1 or more test lines will appear. Proceed with a confirmatory test



Invalid test result = repeat testing required

The test is invalid if 1 or more control lines do not appear. Repeat the test with a new DrugWipe device

*Results are valid for the following 10 minutes after the test has run

